

PTC Needle -Chiba Needle-

Ultrasonically guided percutaneous transhepatic cholangiography (PTC) -B Needle




Intended for Use

Used for Percutaneous Nephrostomy procedures, Cytological biopsies, needle-aspirations, thoracentesis exploratory punctures, to inject contrast dyes. Percutaneous transhepatic cholangiography (PTC).

Description

- Centimeter marked cannula
 - Color code available to identify the accurate size easily
 - Echogenic needle tip design for optimum ultrasound visibility
 - Chiba Biopsy Needle is a universal aspiration needle with depth marks, stopper and echogenic marker placed on the distal tip.
- The Chiba Biopsy needle is a highly versatile needle with many applications. It can be used to inject drugs and contrast agents, as well as for fine-needle aspiration biopsy. Mainly used for cytological and amniotic liquid aspiration procedures, suctions of cyst and alcohol injections.

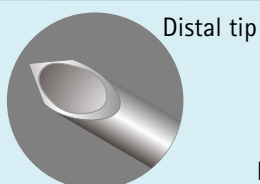

Distal tip

Marking per 1cm

Rough surface processed on both part of cannula and stylet 5cm distance from tip

Stopper for depth marking

Hub*

Stylet Hub*

* color code

Specification

* below 8 digits is product code

| Gage | Color code | Length : 150 mm | 180 mm | 200 mm |
|------|------------|-----------------|----------|----------|
| 18 G | Pink | 22411810 | 22411820 | 22411830 |
| 19 G | Brown | 22411910 | 22411920 | 22411930 |
| 20 G | Yellow | 22412010 | 22412020 | 22412030 |
| 21 G | Green | 22412110 | 22412120 | 22412130 |
| 22 G | Black | 22412210 | 22412220 | 22412230 |
| 23 G | Right-Blue | 22412310 | 22412320 | 22412330 |

Packaging

5 units per Box, each pillow package, Eto sterilized.

Caution

Please read carefully enclosed instruction for use prior to use.

1. Wire of outer 0.45mm is not able to insert to 22G PTC needle. Please use PTC-D set T-type 2 step puncture needle size 22Gx180mm, when use 0.45mm wire.
2. This product is single-use, do not re-use.
3. Specification may be changed without notice.
3. Do not use for the therapy out of intended for use.