

Disposable Endometrial Suction Sampler

ENDOSUCTION





Indications


For examination uterine contents

Description


- The device consists of a clear, flexible outer sheath 250mm and 3.0 mm OD (outside diameter), marked with 9 sounding marks.
- Distal Type are available Open or Closed depending on a physician and patient's situation.
- The device draws cell through a distal collection port by means of a vacuum, created when the inner plunger is pulled through the outer sheath.

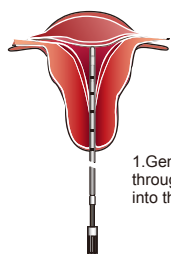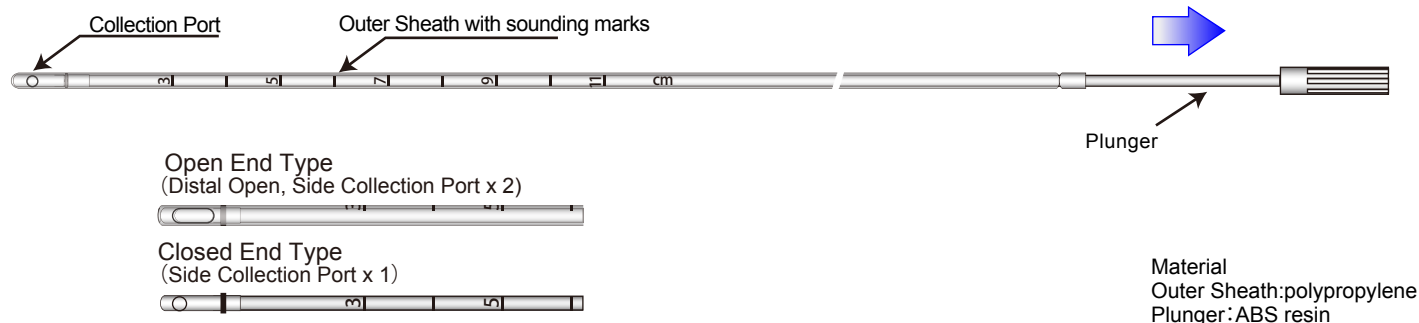


1. Gently admit the ENDOSUCTION through the cervical canal into the uterine cavity.

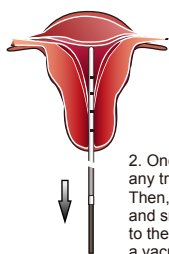

2. Once in position, discontinue any traction applied. Then, hold the sheath in one hand, and smoothly draw back the plunger to the stop, thus creating a vacuum within the sheath.

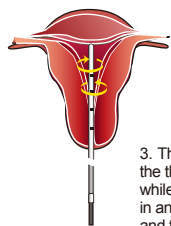

3. The sheath is then rotated between the thumb and forefinger, while at the same time it is gently stroked in and out between the internal cervical os and the fundus of the uterus.

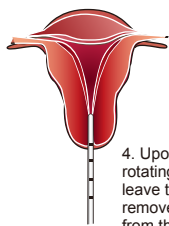

4. Upon completion of the rotating/stroking motions, leave the plunger withdrawn and remove the entire device from the canal.

【Endometrial Specimen Collection】

- 1) Open End Type
Express the contents
- 2) Closed End Type
Snip the distal end of the sheath immediately proximal to the distal opening with a sterile pair of scissors.

SPECIFICATIONS

TYPE and SIZE

Open End Type 2.5×3.0×250mm

Closed End Type 2.5×3.0×250mm

Package

10 pcs/BOX with EOG sterilized

Expired date on Label

•Please read carefully IFU before use.