


Suture-passing Needle

LAPA-HER-CLOSURE™

Approval No. : 21900BZX00941000

passer, wire/ligature, single-use, controlled medical device

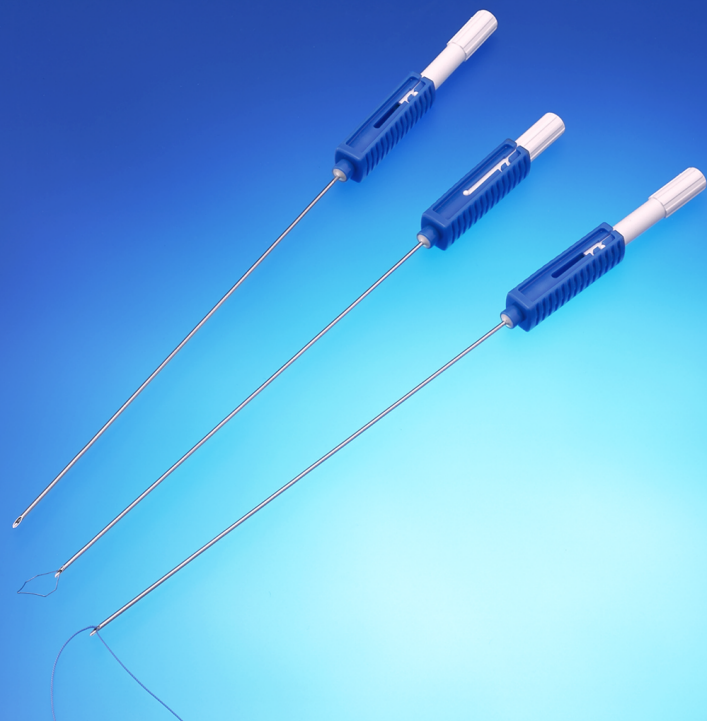

Usage


To guide suturing during laparoscopic pediatric inguinal hernia repair (LPEC method)

Characteristics

- The outer diameter of LAPA-HER-CLOSURE (needle tube) is 19G, which accommodates a suture of size 2-0.
- The loop assembly for suture grasping at the tip of the inner needle enables the user to grasp a suture with a simple maneuver. A suture can be released easily.
- The needle can be inserted transcutaneously to the target site while holding the suture.

LPEC method: Laparoscopic percutaneous extraperitoneal closure

• Please read the package insert that comes with this medical product before use.
 • The package insert describes important information including the method of use and precautions.

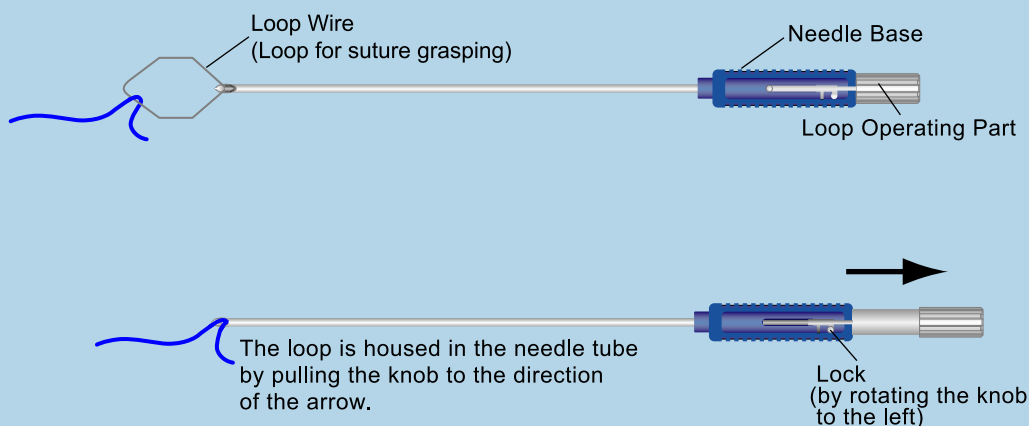

Method of Use

1. Take the product out of the package. Remove the protector. The product is packaged with the loop wire outside of the needle tube.
2. When the needle is inserted into the body while holding the suture, pass the suture through the loop and fully pull the knob (loop operating part) to house the loop wire inside the needle tube. Rotate the knob to the left in this position to lock the loop wire.
3. Carefully insert the device via an appropriate access site while monitoring the position of the needlepoint under microscopic observation.
4. When the suture is released in the body cavity, fix the needle base so that the needlepoint does not move. Then, rotate the knob to the right to release the lock and slowly slide the knob.
5. To hold the suture in the body cavity again, grasp the suture with biceps, pass the suture through the loop, and repeat the procedure in 2.

Specifications

Product Code	size
29110910	19G×130mm (Diameter of the loop for suture grasping : 0.21 mm)
29111030	19G X 80mm (Diameter of the loop for suture grasping : 0.21 mm)

Package Form

5 units per box, individual peel-open package, EOG sterilized